
Belt Mounting Extension Kit

INSTALLATION AND USER'S INSTRUCTIONS

ART NO. HKA12-1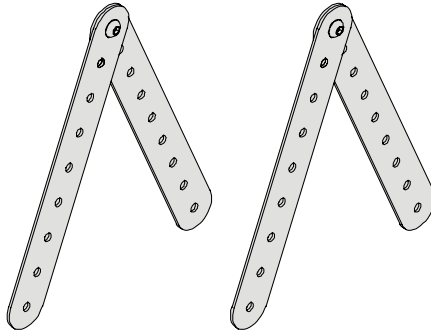

These instructions provide important information for the safe use and maintenance of all Bodypoint Belt Mounting Extension Kits. Give these instructions to the user or their caregiver and review them to ensure that they are understood.


Optional Bodypoint belt mounting hardware may be used for mounting to wheelchairs and seating systems.

This product is intended for mounting positioning belts and harnesses. It is NOT intended for use as a transportation safety device.

WARRANTY: This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.


For more information on Bodypoint products, and a list of distributors outside the USA go to www.bodypoint.com.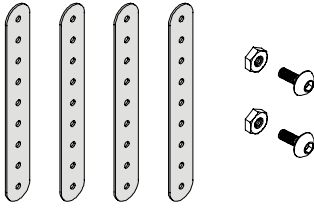


PACKAGE CONTENTS

- 4 Extension Straps
- 2 M6 x 12 Button Head Socket Cap Screws
- 2 M6 Nylock Nuts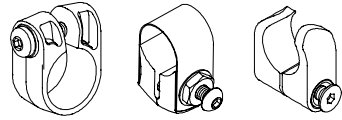


MOUNTING OPTIONS


Extension straps can be attached to the wheelchair using FrameSaver™ Clamps, Band Clamps, or Frame Clamps. Direct mount to the frame, or solid seats using M6 (1/4") screws (not included in package).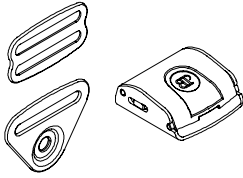

The included M6 Screws and Nylock nuts work with Flat-Mount™ and Cinch-Mount™ End-Fittings.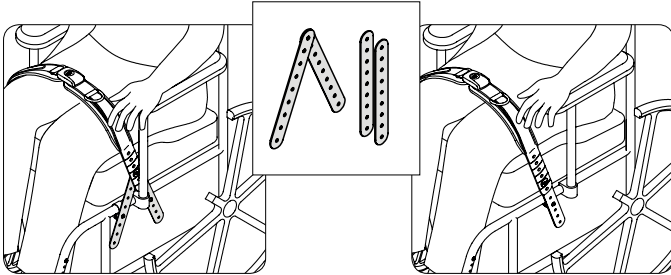


Evoflex® Pelvic Stabilizer extension.

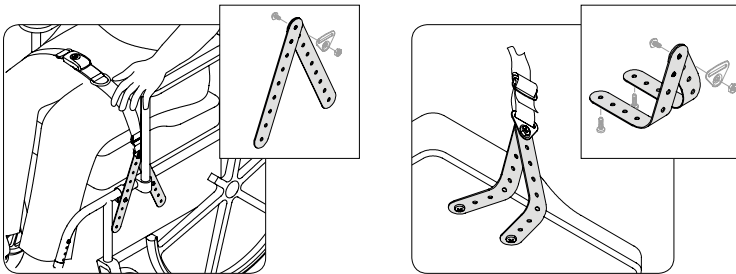Straps spread apart to span obstructions.

Extension straps can be bent to attach directly to solid seat bottoms. Mounting screws must have minimum pullout strength of 90kg (200lbs) (not included in package). Cut extension strap to desired length.

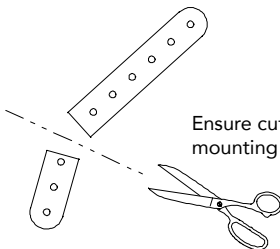Ensure cut location is as far away as possible from mounting hole being used.

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